



PHILIPS
RESPIRONICS
SimplyGo Mini

Reliability that's
simply portable

© 2015 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.
Trademarks are the property of Koninklijke Philips N.V.
or their respective owners.

Broudy HT 5/XX/15 MCI 4106704 PN 1122620

Philips Respironics
1010 Murry Ridge Lane
Murrysville, PA 15668 USA
800-345-6443 · 724-387-4000

www.philips.com/respironics



Built for the demands of today's **active patients**



The SimplyGo Mini portable oxygen concentrator (POC) is designed to help you grow your business by giving you the ability to offer light weight, attractive features, and true reliability that today's patients demand in a POC.

At Philips Respiration, we have earned our reputation for designing and building excellence into all of our products. Now, with SimplyGo Mini, you can also offer your patients excellence in lightweight portable oxygen concentrators.

Reliability matters to your business


When you provide a patient with a POC, you need to know that it's going to be reliable and rugged, helping your business to:

- Eliminate recurring oxygen delivery costs and cylinder hassles
- Reduce maintenance costs
- Ease inventory management
- Minimize patient calls

SimplyGo Mini has undergone the same rigorous testing regimen as that of our SimplyGo POC. SimplyGo Mini was subjected to real-world conditions including impacts, vibrations, and temperatures to deliver reliable performance day in and day out.

Meet more patients' needs with nearly 20% more oxygen

With up to 9 hours battery* autonomy and the ability to deliver nearly 20% more oxygen output than the nearest competitive POC, SimplyGo Mini has the highest oxygen output of any POC weighing 5 lbs (2.3 kg) or less.¹ This means that you can offer more patients the features and performance that they demand from a POC.

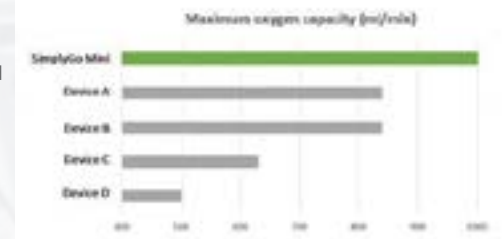

¹9hrs battery autonomy using the extended battery at setting 2, 20 BPM

Why purchasing a quality product matters



Potential increase in patient retention



Potential increase in clinician confidence and referrals



Increase in employee value-added activities because of fewer failure calls



Decrease in inventory because of fewer swap-outs



Decrease in resources and overtime payments to deal with failed equipment calls and pick-ups



Decrease in day-to-day business disruptions because of product failures

Simply smart design

As the smallest and lightest POC ever developed by Philips Respironics, SimplyGo Mini is built for today's patients with features like:

- Attractive and sleek design
- Easy-to-remove long lasting external battery
- Lightweight device (5.0 lbs., 2.3kg)
- Intuitive, easy-to-read screen
- Strong, comfortable carrying case



Product specifications

Oxygen concentrations ²	87 - 96% at all settings ² Based on atmospheric pressure of 14.7 psia (101 kPa) at 70° F (21° C)		
Outlet pressure	20 psig		
Flow settings and pulse volumes	Setting	Nom. pulse vol. at 20 BPM	Max. minute vol. output
	1	11 ml	220 ml/min
	2	22 ml	440 ml/min
	3	33 ml	660 ml/min
	4	44 ml	880 ml/min
	5	50 ml	1,000 ml/min
	+/- 25% or 6 ml, whichever is greater (average of 20 consecutive pulses) over the rated environmental range		
Sound level	43 dBA typical at settings 2 and 20 BPM; 49 dBA typical at settings 5 and 20 BPM		

Physical characteristics

Size	With standard battery: 9.4 in. x 8.3 in. x 3.6 in. (23.9 cm x 21.1 cm x 9.1 cm) With extended battery: 10.2 in x 8.3 in x 2.6 in (25.9 cm x 21.1 cm x 9.1 cm)		
Weight	5 lbs. (2.3 kg) with standard battery installed 6 lbs. (2.7 kg) with extended battery installed		
User interface	Touch screen LCD		

Operating conditions

Operating temperature	41° F to 95° F (5° C to 35° C)		
Relative humidity	15% to 93%		
Altitude	Up to 10,000 ft. (3,048 m)		

AC power

Input	100 to 240 VAC, 50/60 Hz		
Output	19 VDC, 6.3 A max		
Power consumption	120 W while charging		

DC power

Input	12 - 16 VDC		
Output	19 +/- 5% VDC, 6.3 A		

Audible alarm

Sound level	56 dBA or greater when measured at 1 m		
Alarm type	No breath, low oxygen concentration, high breath rate, technical fault, low battery, no flow, depleted battery, external power failure		

Battery

Chemistry	Lithium ion 14.4 VDC (nominal)		
Battery options	Standard battery, Extended battery (equals two standard batteries)		
Dimensions	Standard battery: 7.9 in. x 3.6 in. x 1.0 in. (20.1 cm x 9.1 cm x 2.5 cm) Extended battery: 7.9 in. x 3.6 in. x 1.9 in. (20.1 cm x 9.1 cm x 4.8 cm)		
Weight	Standard battery: 1.1 lbs. (0.50 kg) Extended battery 2.1 lbs. (0.95 kg)		
Watt-hour rating per standard battery	3.4 Ah x 3.6V x 8 cells = 97.9 Wh per standard battery		
Charge time	Standard battery: 4 hours maximum Extended battery: 8 hours maximum		

Typical battery duration on pulse setting 2, 20 BPM³

Standard battery	Up to 4.5 hours
Extended battery	Up to 9 hours

³Battery duration times are based on new, fully charged batteries used with a new SimplyGo Mini System. Battery duration times will degrade with battery age, environmental operating conditions, use over time, and operational condition of the concentrator.

Ordering information

Device	Part number
SimplyGo Mini system with standard battery includes: SimplyGo Mini portable oxygen concentrator, carrying case, one standard rechargeable lithium ion battery, DC and AC power cords, accessory bag, and user manual	International: 1113604
SimplyGo Mini system with extended battery includes: SimplyGo Mini portable oxygen concentrator, carrying case, one extended rechargeable lithium ion battery, DC and AC power cords, accessory bag, and user manual	International: 1113605



Replacement accessories	Part number
Standard lithium ion battery	1116816
Extended lithium ion battery	1116817
SimplyGo Mini AC power supply	1116818
SimplyGo Mini DC power supply	1116819
SimplyGo Mini External battery charger	UK plug 1119949
SimplyGo Mini External battery charger	EU plug 1119950
SimplyGo Mini airline cable	1113902

Accessories - Bags - Brown	Part number
SimplyGo Mini carry bag and strap, brown	1119897
SimplyGo Mini accessory bag, brown	1116824
SimplyGo Mini backpack, brown	1116836

Accessories - Bags - Black	Part number
SimplyGo Mini carry bag and strap, black	1119928
SimplyGo Mini accessory bag, black	1119893
SimplyGo Mini backpack, black	1119894

Warranties	Part number
SimplyGo Mini 2-year sieve warranty extension	1120123
SimplyGo Mini 3-year system, 3-year sieve warranty	1122070

Learn more

For more information about SimplyGo Mini, visit www.philips.co.uk/simplygomini